**H100**  
**H105**  
**H110 - H111 - H112**  
**H190**  
**1930E**  
**H123 - H124 - H125**  
**H126**  
**H320**  
**H130**  
**H330/1**

- F** 1. *Guide utilisateur*  
**GB** 2. *User's manual*  
**D** 3. *Gebrauchsanweisung*  
**E** 4. *Manual del usuario*  
**I** 5. *Manuale dell'utente*  
**NL** 6. *Gebruiksaanwijzing*  
**P** 7. *Guia do utilizador*  
**S** 8. *Bruksanvisning*  
**DK** 9. *Brugsanvisning*  
**FIN** 10. *Käyttäjän käsikirja*  
**N** 11. *Bruksanvisning*

## 1. FRANCAIS

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## 2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

**The following operating and maintenance instructions must be observed:**

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

### BATH SEATS

#### H100 seat with suckers

Adjust the width of the feet to that of the bath using the plastic screws under the seat. Check that there are four suckers before placing the seat inside the bath. The seat can be used as a footstool outside the bath.

#### H105 suspended seat

Place the bath seat's armrests over the sides of the bath. You can remove the back if required (when used with a H110, H111 or H112 bath board).

### BATH BOARDS

#### H110 board

- Place the board over the sides of the bath and adjust the sides to suit the width of the bath (using the plastic screws under the board).
- Check that the board is stable and that the white, non-slip feet are in good condition before use.

#### H111 board

Please note, when unpacking the product, the feet are stored under the right and left-hand flaps in the box.

- Insert the 4 feet into the circular holes under the board and slide them into the retaining slots.
- Place the board over the sides of the bath and adjust the feet to its width. Screw the feet to tighten them.
- Check that the board is stable before use.

#### H112 board

- Insert the 4 feet into the circular holes under the board and slide them into the retaining slots.
- Place the board over the sides of the bath and adjust the feet to its width. Screw the feet to tighten them.
- Check that the board is stable before use.

### H190 NON-SLIP BATH MAT

Place the mat in the bath, sucker side down.

### 1930E BATH ACCESS BAR

- Position the access bar over the sides of the bath with the sliding feet inside. Tighten the plastic screws.

- Check that the bar is installed correctly and that the black non-slip feet are in good condition before use.

### H123, H124, H125, H126, H130 AND H320 PLASTIC SUPPORT BARS

Place the support bar at the required location and mark the position of the fixing holes. Drill and fix with the appropriate plugs.

### H330/1 FOLD-AWAY SUPPORT BAR

Position the support bar beside the toilet. Fix to the wall with four 5x50mm screws and plugs.

### MAINTENANCE

Clean regularly with a normal household cleaner. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

### USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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