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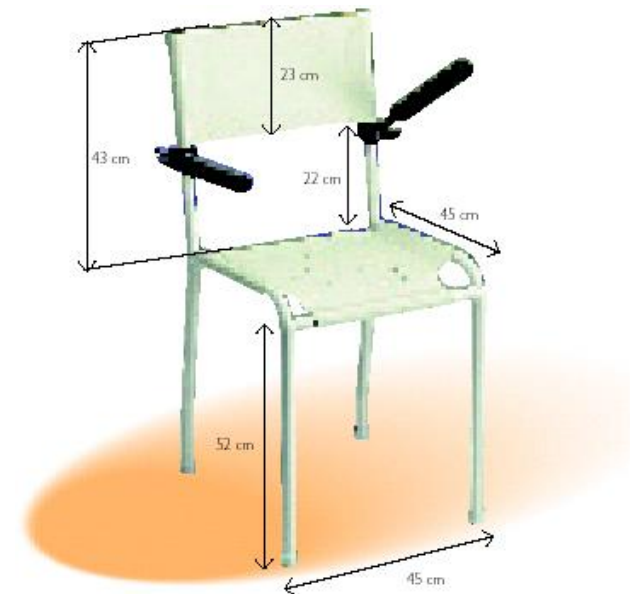
Production/ sales / logistics/ administration

Invacare Limited  
Pencoed Technology Park, Pencoed  
Bridgend CF35 5AQ  
Tel: (44) (0) 1656 776200  
Fax: (44) (0) 1656 776201  
[UK@invacare.com](mailto:UK@invacare.com) / [www.invacare.co.uk](http://www.invacare.co.uk)

# General

## SHOWER SEAT

The shower seat provides an aid to washing and the care of patients in wet cells.

The following shower seat models are available:

- Shower stool (no backrest); type 7701
- Shower seat (52cm in height) with fixed armrests; type 7707
- Shower seat (48cm in height) with fixed armrests; type 7707,048
- Shower seat with detachable armrests; type 7708
- Shower-toilet seat with fixed armrests, polyurethane seat, rail for bedpan or bucket; type 7713
- Models 7708 and 7713 are also available with separately folding stainless steel armrests (option 73).

## Introduction

We at Invacare are delighted that you have chosen to purchase a quality aid.

A long history of developing and manufacturing bathroom, shower and toilet aids, has enabled Invacare to become a leading company in this field.

The shower seats satisfy the CE safety standards. These standards are in compliance with the medical devices directive of 14 June 1993 no. 93/42 EEC.

## Warranty

All Invacare toilet aids come with a 2-year manufacturer's warranty. Normal wear and tear and improper use do not fall under the terms of this warranty.

Invacare does not accept any liability for personal injury which is caused by the improper use of the shower seats.

## Safety

The shower seat is designed to be used safely and easily, primarily in wet cells.

**Please note: When raising / lowering the armrests (option 73), take care not to trap your fingers in the hinge point.**

In the event of queries or defects, please contact Invacare.

## Maintenance

The shower seat and optional folding armrests (73) can be rinsed down after use with water and normal household detergents; do not use a scourer or aggressive solvents for this purpose.

## Folding the armrests (option 73)

The separately folding armrests can easily be lowered and raised. When in their raised position, the armrests should remain firmly in position. If not consult your dealer.

# Technical specifications

- Steel frame with shockproof and impact-resistant Rilsan coating.
  - Polyurethane folding armrests (option 73).
  - Maximum capacity: 135 kg.
  - Dead weight (basic model)
- |           |         |
|-----------|---------|
| 7701:     | 4.4 kg  |
| 7707:     | 7.4 kg  |
| 7707,048: | 7.4 kg  |
| 7708:     | 7.4 kg  |
| 7708,073: | 8.0 kg  |
| 7713:     | 14.2 kg |
| 7713,073: | 14.8 kg |