Invacare® H304 Finesse

en	Raised toilet seat User Manual	1
de	Toilettensitzerhöhung Gebrauchsanweisung	3
fr	Siège de toilettes rehaussé Manuel d'utilisation	5
it	Rialzo WC Manuale d'uso	6
es	Asiento elevador de inodoro Manual del usuario	7
pt	Elevador de sanita Manual de utilização	9
nl	Verhoogde toiletzitting Gebruiksaanwijzing	10
sv	Upphöjd toalettsits Bruksanvisning	12
fi	Kohotettu wc-istuin Käyttöohje	13

en

I General

I.1 Images

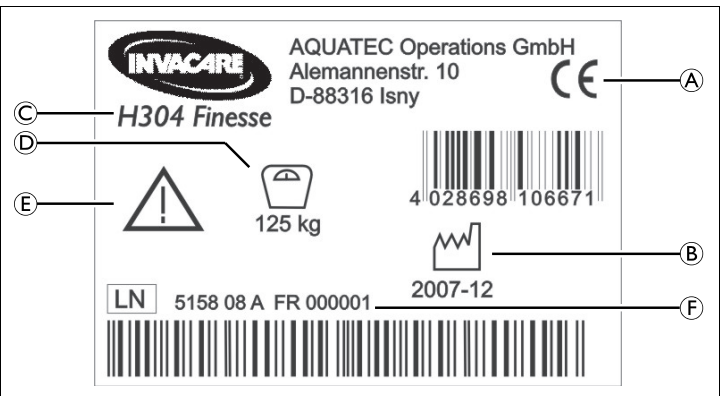
Fig. 1

Fig. 2


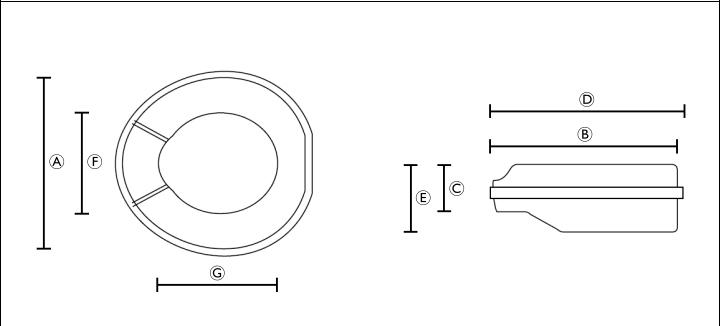
Fig. 3

Fig. 4

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.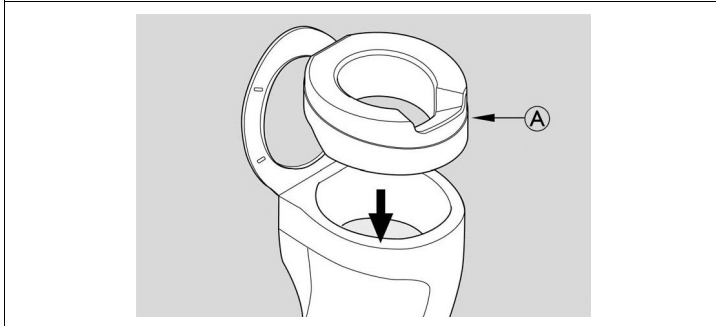

Yes, you can.®

1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

1.3 Intended use

The product serves exclusively to raise the toilet seat and is therefore only intended for installation on toilets. Any other use is prohibited.

Operating temperature: 10 – 40 °C

1.4 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

(A)	Raised toilet seat
(B)	User manual

1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Quality

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. It is Invacare's goal to ensure that we, at the widest possible range, use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

1.6 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING!

Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use for transfer from the side or partial load.
- Do not sit on the product if not all four legs have full contact to the ground.
- Do not tilt.

WARNING!

Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.

2.2 Labels and symbols on the product (Fig. 2)

(A)	Conformity mark
(B)	Product time frame
(C)	Product designation
(D)	Load capacity
(E)	Note (observe accompanying documents)
(F)	Serial number

The identification label is attached to the backside of the product.

3 Assembly (Fig. 3)

CAUTION!

- Make sure that the hygiene recess (A) is facing forwards.
- Make sure that the raised toilet seat is placed even and firmly fits on the toilet bowl.

1. Place the raised toilet seat directly on the toilet bowl.

4 Maintenance

4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely attached to the toilet bowl.
- Regularly clean and disinfect the product.

4.2 Cleaning and disinfection

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CGI39 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 60 °C.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

5 After Use

5.1 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (© 4.2 Cleaning and disinfection, page 3), check the product for damages and ensure a proper mounting.

5.2 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

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Invacare Austria GmbH
Invacare AB
Aquatec Operations GmbH